





Feebris Guide

For Norfolk & Waveney GP Practices





Digital Care Homes and Social Care Team

Table of Contents

What is Feebris?	01
Setting up your Feebris account	02
Accessing residents	03
Adjusting resident's NEWS2 baselines	05
'Filter' feature	06
Further support	08





What is Feebris?

The NHS Norfolk & Waveney ICB is currently running a pilot remote observations programme with some of your local care homes.

Patient Test

Feebris is a mobile app and observation kit that enables Care Home staff to conduct a check-up, capturing observations from Bluetooth connected medical grade sensors.

The Feebris app then allows users to triage their check-ups using established NEWS2 guidelines and take appropriate action.

Clinicians/GPs also have access to patient checkups on the Feebris portal. Here, they can review observations and adjust patient's NEWS2 baselines to establish Personal Early Warning Scores.

Feebris helps medical professionals to triage patient check-ups to ensure that escalation happens when appropriate.

CHECK-UP RISK LEVEL REQUIRES CHECK-UP RISK LEVEL REQUIRES												
PR 96	^{SP02} 96	rr 17										
NEWS:1	NEWS:0	NEWS:0										
A News:0	bp 132/86 News:0	TEMP 38.0 °C News:0										
More Details QUESTIONNAIRE												
Vomiting/nause	a Normal fluid i	ntake										
Weight loss												
,	four Next Action	Weight loss										

National Early Warning Score

Ļ	Dashboard Virtual Ward Manage Patients										
	Patient 1 Test · Female, 69 y.o. · Demo Organisation, 123 road Latest Check-up Feb 7th 2023, 3:01 pm NHS Number: -										
≓¥ VIEW	EW Personal EWS National EWS										
Date	Date Feb 07 3:01 pm										
eathing Rate	20	>= 38									
SpO ₂	97	34 - 35									
BP Syst	138	32 - 33 30 - 31									
Pulse Rate	106	28 - 29									
Temperature	38.1 °C	25 - 25 22 - 24									
nsciousness	C	21 - 21									
RP Dise	96	18 - 20									



Setting up your Feebris account

Once your local admin team have created, your Feebris account you will receive an email to complete your account set up. The email will have a link to reset your password.

The email will have a link to reset your password.

Due to the nature of the data Feebris holds, the platform uses the secure **"3 random words"** password system, where a password will be generated for you to use.

You can cycle through different randomly generated word combinations until you find a memorable one, before clicking **"Save Password"**. You can then use your email and this password to login.





Accessing residents

After logging in, you'll see a list of residents assigned to your GP practice, along with their most recent set of NEWS2 observations. This list is managed by the local care homes directly. Click on a residents name to see more information about them.

S Virtual Ward								¢ 6
2 m - 1	Latert Chark-on J.	NEWS / EWS Total	Breathing Data	Su()2		Pulse rate		III 🕹
	Catest Crieckrup ψ	Score	Breating Rave	spoz U	Biodu Pressure 0	Puise rate	Consciousness 0	Temperature 0
	· · · · · · · · · · · · · · · · · · ·	-	Ŧ		Ŧ	Ŧ	Ŧ	-
NH5 004 0	Jun 27, 11:14 am	0	19	98	170/105	81	Alert	36.6 *C
Patient, Test	Jun 27, 8:46 am	-	-	-	-	-	-	38.9 °C
NHS Number: 418	Jun 26, 8:57 am	-	-	-	107/70	74	-	36.7 °C
NHS Number: 604	Jun 22, 1:35 pm	3	12	98	96/66	108	Alert	36.7 °C
NHS Number: 458	Jun 22, 1:22 pm	2	12	95	136/84	93	Alert	37.2 °C
NHS Number: 494	Jun 21, 5:20 pm	-	14	98	121/50	56	-	36.8 *C

A residents page will have all sets of observations saved to their record, displaying 10 at a time starting from most recent. You can click on the date to bring up the full recorded details from that observation check up, this will include any stethoscope recordings, pictures taken and notes left by the care home staff.

Date	Jun 16 5:08 pm	Jun 19 4:09 pm	Jun 19 5:19 pm	Jun 20 4:45 pm	Jun 21 11:30 am	Jun 21 5:16 pm			
Breathing Rate	16	15		16	14	14			
SpO ₂	95	96		91	97	98			
BP Syst	149	82		93	110	121			
Pulse Rate	79	66		52	53	56			
Temperature	36.8 °C	36.4 °C		36.5 °C	36.5 °C	36.8 °C			
Consciousness									
BP Diast	101	45		41	54	50			
Total Score (Personal EWS)	N/A	N/A	N/A	N/A	N/A	N/A			
Advised Action	N/A	N/A	N/A	N/A	N/A	N/A			
Selected Action	Monitor	GP	Monitor	Monitor	None	None			



Digital Care Homes and Social Care Team





To change the baselines that the NEWS2 scores against for individual residents, click the "Edit EWS" button. This will bring up a prompt confirming that changes made to the Early Warning Score parameters will affect the score given and recommended action.

Each of the 6 parameters then has draggable sections on that can be adjusted. The right column is the National Standard, the left is the personal one for that resident. Each parameter can be adjusted separately, before clicking "Save" at the top.



Adjusting resident NEWS2 baselines

After logging in, you'll see a list of residents assigned to your GP practice. Using the search bar and either a name or NHS number, search for resident you would like to change the baselines for, click on their name and then click "**View Patient Record**".

Virtual Ward All Patients -								1 mi	Updated nute ago C	Þ
Sorted by Last Vitals 👻	All 👻	Test					25 F	Rows 👻 Ik	< 1	> >1
Test Patient -	?	Custom Checkup Jun 28th 2023, 1:03 pm	BP 115/80	Sp02 -	RR -	Pulse 81	Temp 36.9 °C	LOC	Action None	^
No current admission						v	IEW LATEST CH	ECKUP	IEW PATIENT R	ECORD
test patient	2	Custom Checkup	BP	Sp02	RR	Pulse	Temp	LOC	Action	



Each of the 6 parameters then has a draggable blue bar, adjusting these will adjust what an observation result will score as. These parameters are displayed vertically, you may need to scroll down to see them all.

The left column is the individual parameters that can be adjusted, while the right column is the national standard which are displayed for reference.

Each parameter can be adjusted separately, before clicking "**Save**" at the top. Only observations taken after the changes have been saved will be based on the updated parameters. To change the baselines that the NEWS2 scores against for the resident, click the "**Edit EWS**" button, and agree to the prompt that appears, confirming you understand adjusting these may affect the NEWS2 score generated which will impact the escalation pathway suggested.





'Filter' feature

When you login to the <u>Feebris Portal</u> you will be directed to the Virtual Ward landing page.You will find a "Filter" button on the page. Click on it to display the 'list of care homes you are linked to:

Virtual Ward Dash Monitor patients in your wa	board Irds									Updated 16 seconds ago C II
Filter = Sort 1						•				10 Rows - IC - C 1 -> ->I
										C Missed checkup 1 month ago
Abigale Herzog –	1	Routine Checkup Nov 19th 2023, 10:15 pm	BP 82/38	Sp02 97	RR 10	Pulse 162	Temp 38.1	LOC P	Action None	~
Augustus St. Cloud 943 580 7194	11	Basically Routine Checkup Apr 15th 2024, 2:56 pm	BP 90/60	Sp02 80	RR 25	Pulse 120	Temp 38.0	LOC A	Action None	~
Bob Sharp -	3	Routine Checkup Nov 19th 2023, 10:01 pm	BP 88/157	Sp02 66	RR 10	Pulse 218	Temp 39.0	LOC P	Action 999	~
Nicki Manaja 944 109 9708	1	Routine Checkup Apr 3rd 2024, 6:37 pm	BP 213/145	Sp02 47	RR 7	Pulse 132	Temp 38.0	LOC V	Action GP	~
Billy Quizboy	?	AKI Questions Checkup Apr 15th 2024, 12:02 pm	BP -	Sp02	RR -	Pulse	Temp	LOC	Action None	~

From the list, you can select or deselect one or more care homes to tailor your dashboard view. This allows you to focus specifically on residents from particular care homes.

*Tip: If you are linked to multiple homes, you can type in the "Care home" field to search for a specific care home by name to narrow down the list.



Clear Selection: If you wish to remove a care home from your view, simply click the small "X" next to the care home name at the top of the dashboard.





Additional information: the process for linking a resident with a GP clinical organisation remains unchanged. For a refresher on assigning a GP practice to a resident's profile, please refer to our existing guide:

Filters	Reset Filters
Name or NHS number	
All vitals history	•
Care home	-







Further support

You can seek further support & help through the Feebris dashboard.

On the dashboard, if cookies have been enabled, there will be a chat icon in the lower right hand side. You can click on this to bring up a chat option direct with Feebris who can help with your request.

Feebris can also be emailed directly at support@feebris.com



Alternatively, you can contact the NWICB Digital Care Homes and Social Care Team via MS Teams by clicking here to start a conversation, or you can emailing us: nwicb.digitalsocialcare@nhs.net